TIK'ALERT® Self-test

Ref. 70084BI
Self-diagnostic test for the detection of anti borrelia IgM antibodies in whole blood in case of tick bites.

GENERAL POINTS

Ticks are insects living in undergrowth and forests of temperate climatic zones. Those insects can be infected by a bacterium (Borrelia) that can be then transmitted to humans (1 to 2% transmission risk) after being bitten on the arm, the leg and any other part of the body not protected with clothes. Borrelia bacterium affects the nervous system and can lead to serious neurological disorders. European strains of Borrelia include B. afzelli, B. garinii, B. burgdorferi sensu stricto B. spielmanii, and B. bavariensis which are known pathogenic. Species B. valaisiana and B. lusitanae are potentially pathogenic. That infection can also cause cutaneous lesions and arthritis. From 3 to 10 days, or even several weeks, after being bitten, an inflammation and redness (erythema) can appear in the bite region. The inflammation can be accompanied by feverishness. In case of infection, the immune system reacts by releasing specific antibodies directed against Borrelia bacterium. A particular category of antibodies (IgM: immunoglobulins M) which appears first can be detected between 2 and 6 weeks after being infected. Lyme disease develops in 3 stages: In phase I (2 to 4 weeks), The IgM antibodies appear only in 40 to 60 % of the cases. During the phase II (4 to 6 weeks later), the antibodies are detectable in 70 to 90 % of the cases. At last during phase III, the antibodies are generally present and can be detected. Using TIK'ALERT® test, it is possible to detect the presence of those specific IgM antibodies which appears first, and thus, a recent infection with Borrelia bacterium between 2 and 6 weeks after the presumed tick bite. Should the case arise, an antibiotic therapy will eliminate the infection.

Attention: However, this screening test cannot detect the stage of the illness in case of positive results.

Questions and answers

How does TIK'ALERT® work?

In presence of Borrelia bacterium transmitted by a tick, specific IgM antibodies are produced first by the immune system. The antibodies circulate in the blood and bind on the encountered bacteria. The Borrelia antigens coated on the TIK'ALERT® strip specifically detect those antibodies by producing a coloured line under the T mark of the cassette. A control line capturing the reagent excess appears as a coloured line under the C mark of the cassette. In case both lines appear, the presence of Borrelia bacterium can be suspected.

When should the test be used?

TIK'ALERT® test should be performed between 2 and 6 weeks after being bitten by a tick or as soon as a round inflammation (erythema) appears on a part of the body likely which was exposed to a tick bite. It corresponds to the phase 1 of the disease for which the TIK'ALERT® is well suitable and for which the test sensitivity is 53.8%. Contamination happens only in 1 to 2% of the cases, but the presence of IgM antibodies, detected thanks to a positive result, indicates an infection that should be rapidly treated.

Can the result be incorrect?

The results are accurate as far as the instructions are carefully respected. Nevertheless, the result can be incorrect if TIK'ALERT® test gets wet before test performing or if the quantity of blood dispensed in the sample well is not sufficient. The plastic pipette provided in the box allows making sure the collected blood volume is correct.

How to interpret the test if the colour and the intensity of the lines are different?

The colour and intensity of the lines have no importance for result interpretation. The lines should only be homogeneous and clearly visible. The test should be considered as positive whatever the colour intensity of the test line is.

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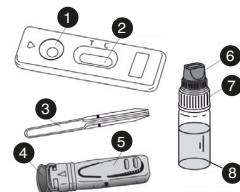
Presentation

The box contains the material needed to perform a test:

- 1 sealed aluminium pouch containing:
- 1 test device
- 1 plastic pipette
- 1 dessiccant pouch

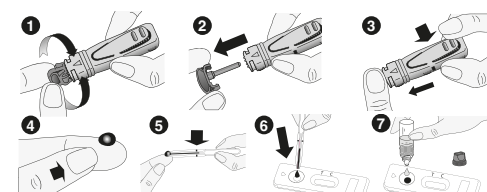
Only open the protective pouch when you are ready to use the test. The dessiccant pouch should not be used.

- 1 sterile lancet for blood sampling (+ one extra)
- 1 diluent vial of 1 mL of diluent.
- 1 instruction leaflet

Precautions

1. This test is exclusively intended to in vitro diagnostic use. External use only. DO NOT SWALLOW.
2. Carefully read the instructions before performing the test. The test is only interpretable if the instructions are carefully respected. Follow strictly the indicated blood and diluent quantities, as well as reading times.
3. Store between +4°C and +30°C. Do not freeze the test.
4. Do not use after the expiry date printed on the label and on the protective pouch or if this pouch is damaged.
5. Do not reuse the TIK'ALERT® test.
6. Keep out of reach of children.
7. After use, all the components can be discarded in a dustbin.

EN

PROCEDURE:

Testing procedure always starts with a good preparation.

Place the content of the box on a clean, dry and flat surface (e.g. table). Then the testing follows:

1. Wash your hands thoroughly. Use soap and warm water. Dry your hand with clean towel.
2. Prepare the test device and the pipette. Take them out from the protective pouch (tear at the notch) and place them in the reach of your hands (you will need them later). Dispose the small dessiccant bag.
3. Prepare the lancet. Twist off and discard lancet's grey cap (image 1 & 2) without touching the trigger button.
4. Clean the end of the forefinger or of the middle finger with cotton damped with alcohol. Rub the chosen finger towards the tip to later on enhance the blood stream.
5. Press platform firmly against the previously cleaned finger, and press the release trigger button (image 3).
6. The tip will automatically retract into the body of the device.
7. Rub the finger's end to obtain enough whole blood sample (image 4).
8. Without pressing the bulb, put in contact the plastic pipette with the blood sample. The blood migrates into the pipette through capillarity to the line indicated on the pipette. You may rub again your finger to obtain more blood if the line is not reached. As far as possible, avoid air bubbles.
9. Put the blood collected with the pipette into the sample well of the device, by pressing on the pipette bulb (image 6).

What is the accuracy of TIK'ALERT® ?

The TIK'ALERT® test is accurate and is used by professionals in the field (hospitals, laboratories...). A study performed on a panel of 251 patients samples shows an overall result correlation of 88% when compared to reference methods. However it is still possible that the IgM antibodies appearing after an infection by Borrelia are not detected due to late maturation of the immune system of certain persons depending on the level of the disease. Despite the reliability of the test, falsely positive or falsely negative results are possible. This is especially true for patients being sero positive for cytomegalovirus (CMV) which may show false positive results when using the TIK'ALERT® test.

Information on Borrelia infections and related disease

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EN

Questions and answers

What is the line that appears under the mark C (Control) for?

When this line appears, it only means that the test was performing well.

If I read the result after 15 minutes, will the result be reliable?

No. The result should be read within 10 minutes after adding the diluent. The result is reliable up to 15 minutes.

What do I have to do if the result is positive?

If the result is positive, it means that IgM antibodies directed against Borrelia bacterium are detectable in blood and that you must absolutely consult a doctor showing him the test result. Then, the doctor will order a treatment to follow in order to avoid complications arising after this infection. In case of past infection having occurred several years before, a positive result can be obtained as the antibodies may be persisting in some persons. A positive result can also be obtained in patient having been efficiently cured with antibiotics.

What do I have to do if the result is negative?

If the result is negative, it means that no IgM antibodies directed against Borrelia bacterium are detectable in blood. **Caution:** Falsely negative results can be observed during phase I (antibodies appearing in 40 to 60% of the cases) or during phase II (antibodies rising in 70 to 90% of the cases) of the disease development. In case of negative result it is strongly recommended to perform a new test 2 to 6 weeks later in order to check the increase of antibodies level. If you have been preventively treated by antibiotics, the antibodies are not detectable or cannot be detected. If the tick's bite is too recent (less than 2 weeks), your organism has not had enough time to produce detectable antibodies. It is recommended to repeat the test with a new TIK'ALERT® 2 to 6 weeks after the presumed bite. However, if a rash, accompanied or not by feverishness, persists, it is recommended to consult your doctor.

EN

MANUAL SELF-TEST

TIK'ALERT® Self-test

EN

quicktest.eu

10. Wait 30-40 sec for the blood being totally absorbed into the sample well. **Unscrew the blue cap** of the diluent vial (leave the white cap tightly screwed) and add the diluent as follows:

- Hold the diluent vial vertically and slowly add exactly 4 drops in the sample well** of the device with an interval of 2-3 seconds between each drop.
- 11. Read the result after 10 minutes. Do not interpret after 15 minutes.

Result interpretation

The intensity of the line colour does not have any importance for the interpretation of the test result.

1. Negative result

Only one coloured line appears under the mark C (Control). This result means that no antibodies directed against Borrelia bacterium are detectable in blood.

2. Positive result

Two coloured lines appear in the window under the marks T (Test) and C (Control). The intensity of the line T may be clearer than the intensity of the line C. This result means that IgM antibodies directed against Borrelia bacterium are present in blood and an infection is starting. **You must absolutely consult your doctor.**

3. Non valid result

No line appears or a coloured line appears under the mark T (Test) without any line under the mark C. In this case, it is not possible to interpret the test, which must be considered as non valid. It is recommended to repeat the test with a new TIK'ALERT® device and a fresh whole blood sample.

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